



The Entellus™ PathAssist™ Light Seeker™ confirms the correct entry into the frontal recess and locates the frontal sinus.



Use of the Light Seeker:

- Step one:** Verify the device has been cleaned, sterilized, inspected and functions properly.
- Step two:** Connect the device to a 2.5mm cable using one of the adapters provided (if necessary). Connect the light cable to a 300W Xenon light source with a 50% initial intensity setting. Increase light intensity if needed to increase light output.
- Step three:** Under endoscopic visualization use the Light Seeker instrument to identify pathway to frontal recess. The frontal sinus will transilluminate when the tip of the Light Seeker is within the frontal sinus.
Note: To enhance the intensity of the transcutaneous illumination, it may be necessary to reduce the number of competing light sources (e.g. endoscope or room light) or to reposition the location of the instrument tip.
- Step four:** Endoscopically note the pathway to the frontal recess utilizing direct visualization and anatomical landmarks.

Angle: 78 degrees

Ball Tip: 1.5mm

Simple, reusable and designed for multiple light source connections, the Entellus Light Seeker offers a practical and economical solution for illuminated guidance.



The Light Seeker has a standard ACMI connection. Two converter adapters are provided.

Wolf Adapter

Storz Adapter

Connections:
ACMI, Wolf
and Storz

The Light Seeker instrument will warm at the connection source. Care should be taken while handling to prevent injury to patient and user.

Warnings:

Do not use the device with a xenon light source > 300W and a light cable > 2.5mm. High energy light radiated through the Light Seeker can result in high temperatures in front of the light outlet and at the connection point with the light cable, including the adapters. Use of a higher wattage light source or larger diameter cable may result in burns or permanent tissue damage to the user or the patient.

Allow the Light Seeker to cool for a few minutes before disassembling from the light cable.

CATALOG NO.	DESCRIPTION
IS-100	Entellus PathAssist Light Seeker

Light Seeker Indications for Use:

The PathAssist Light Seeker is intended to locate, illuminate within, and transilluminate across nasal and sinus structures, including the frontal, ethmoid and maxillary sinuses, in adults aged 18 and over. Please review the Instructions for Use for a complete listing of contraindications, warnings, precautions and adverse events. Refer to PathAssist Light Seeker Instructions for Use (IFU) for complete instructions.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.